

INSTRUCTIONS FOR USE

4-Point Wheelchair Restraint

1. Position wheelchair over floor rails and apply brake.
2. Feed the two front non-adjustable straps around wheelchair frame above front wheels and plug tongues into buckles. Push track clip fittings into rails (**Fig.1**).
3. With over centre buckles open feed the two rear adjustable straps around the wheelchair frame near the rear wheels as high as possible and plug the tongues into buckles (**Fig.2**). For Karabiner version, clip the Karabiners in the same position.
4. Pull webbing to tighten straps.
5. Close over-centre buckles to tension (**Fig.3**). The wheelchair is now secure.
6. To release, reverse above operations.

Fig.1

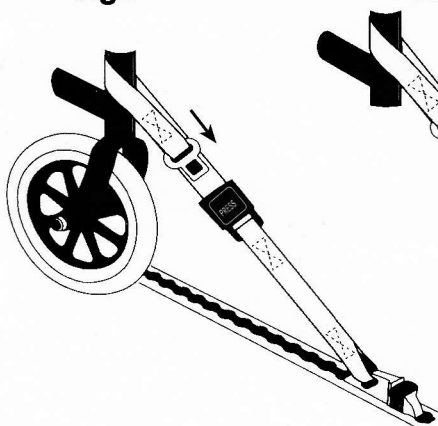


Fig.2



Fig.3

